



Installation Manual

PUPS (Pocket Ultrasound Probe System)

Package Contents:

1. **USB** Ultrasound Probe(s) that was placed with your order.
2. **USB** Memory Stick containing preloaded software, User Manual, and Installation manual.

*** **NOTE:** Please install the software before connecting the Ultrasound Probe ***

Installation:

1. Unpack your new **PUPS** System. (fig. 1)
2. Turn on your Laptop/ Desktop computer. Once your computer system is ready for usage, located the memory stick that the **PUPS** Software is supplied on. (fig. 2)
3. Install the memory stick in to one of your **USB2** (fig's. 3 & 4) ports and follow the on screen instructions.



DMS Software Installation.exe
Direct Medical Systems, Inc.

4. Click on the icon "DMS Software Installation.exe"

The following installation screen will come up:



CLICK **Install Software**



CLICK **Next**



Installation Manual

PUPS (Pocket Ultrasound Probe System)



CLICK **Next**



CLICK **Next**



CLICK **Finish**

5. The Ultrasound Software System is now installed and places a **ICON** on your desktop called **ISSUSB.exe**



6. Click **Finalize and Close**.
7. Remove the memory stick and save it in a safe place.

USB Probe Installation:

1. Connect the **PUPPS** Ultrasound probe to one of your **USB2** ports (**fig. 5**).
2. Windows XP and Vista will display “**FOUND NEW HARDWARE**” message when the probe is inserted. This process will occur twice because of the hardware being installed requires that Windows XP and Vista finds two (2) **USB2** devices within our product and is normal operation. Once this installation process has finished **twice** your system is ready to use.
3. The complete process is shown here in detail and this will repeat a second time.





8. Click Next




9. Click Finish (remember the above 5 steps occurs twice when you first connect a probe.)

System Operational Software:

1. Double click the new icon on your desktop called **ISSUSB.exe** (fig.6)
2. The **PUPPS** software will load and start running on your system. (fig. 7)

Configuration settings:

1. ON the main menu Click **Edit** then Select **Preferences**

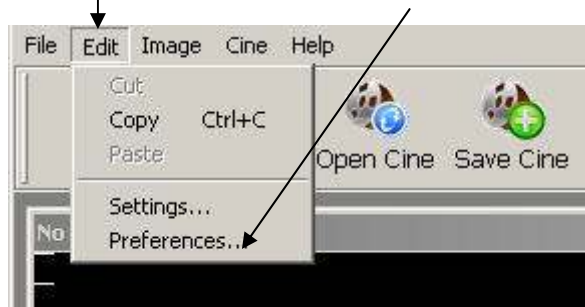



Installation Manual

PUPS (Pocket Ultrasound Probe System)

2. Under **PREFERENCES** dialog box Select all of the options under the **Patient** to be selected (check the boxes). Also do the same under **File load and Save** options. Make your selections look like the following example.
3. Under the **CINE Display** setting, Select the **Frame's** option



Thank you for investing in the PUPS (Pocket Ultrasound Probe System)

If you need any assistance please call Direct Medical System – Technical Engineering Support
Toll Free (888) 600-8666 option 1 – within the USA
Worldwide (925) 600-8463 – option 1